

Instructions for use

Open door → insert Flocare
Infinity pump set
→ close door

NUTRICIA
flocare[↑] Infinity[∞]
Enteral feeding pump
This is a medical device

Clear memory and Prime the line


ON/OFF
hold 2 sec

CLR
volume delivered
will be cleared

FILL SET
hold 2 sec

connect to tube → select  

① Continuous feeding

ml/h on the screen →  400 ml/h
→  1 ml/h

START/STOP



**Default
flow rate = 0 ml/h**

② To set a dose (optional)

ml/h on the screen →  400 ml/h
→  1 ml/h

DOSE = VOL

 4000 ml
→  1 ml

START/STOP

 Manufacturer:
**Nutricia
Medical Devices B.V.**
Taurusavenue 167
2132 LS Hoofddorp
The Netherlands
www.nutriciaflocare.com

Produced by:
ZEVEX® Inc. (USA)



For enteral use only



IPX 5
Type BF

INPUT: 5 V  2,4 A

Internally powered

Patented and patents pending in
Europe, USA and other countries.

CE
0344

27774-001 rev E